

## Instructions for use TD-Box

### Precautions prior to use

- 1 The TD-box is intended for use with rigged and flexible endoscopes in the medical field.
- 2 The TD-box must be placed on a stable underground
- 3 The inside of the TD-box is intended for training. Remove the training inlay of the box before training.
- 4 The hardcover case is intended for transport. Transport is secured by car, boat and airplane.

### Preparation

- 1 Open the TD-box and take out the inlays.
- 2 Put the inner site of the TD-Box at a stable underground
- 3 Use the small introduction inlet for gastroscopes/duodenoscopes and the bigger introduction inlet for colonoscopes
- 4 In case of electrocautery: use the pinch at the metal plate for the backflow of electricity by connecting the cable to the clamp of the electrocautery unit
- 5 Be aware that the cable for electrocautery is placed in the therefore meant niche.
- 6 Do never touch the (metal) inner side of the TD Box during the use of electrocautery.
- 7 Cover the box during training with the transparent or dark cover

### Direction for use

- 1 The TD Box is intended to do training with flexible endoscopes:
  - a. EMR/ESD
  - b. Stenting, self expandable stents
  - c. Foreign body removal
  - d. Bleed and complication management
  - e. Miscellaneous

### Direction after use

- 1 Remove the TD box carefully afterwards your training from the stable underground
- 2 Clean the inside of the box with a 70% alcohol suspension
- 3 Close the TD box for safe travelling
- 4 In case of use with porcine models:
  - a. Clean the box with tissues
  - b. Clean the box with hot water and soap
  - c. Disinfect the TD-box, inside and the outside with a 70% alcoholic suspension
  - d. Put the inside of the TD box back in the hardcover box when it's dry and clean.
  - e. Close the TD Box for safe travelling

### Attention

- 1 Use the TD Box with care during the training.

### Guarantee

The supplier of the hardcase box gives a five year period guarantee at the box. Use in normal conditions.

Medical training tools gives a one year guarantee at the inner side of the box.

**Warning, Warranty, and Limitations:**

- The TD-Box may fail to function for a variety of reasons.
- No representation or warranty is made as the TD-Box will not fail or malfunction.
- The TD-Box is sold in an "as is" condition. All steps in the Instructions for Use are to aid the user in assessing the quality and performance of the TD-Box. The entire risk as to the quality and performance of the TD-Box is borne by the buyer.
- Medical Training Tools disclaims all warranties, expressed or implied, with respect to the TD-Box, including but not limited to any implied warranty of merchantability or fitness for a particular purpose. Medical Training Tools shall not be liable to any direct or consequential damages resulting from the use of the TD-Box or caused by any defect, failure, or malfunction of the TD-Box, whether a claim for such damages is based upon warranty, contract, tort, or otherwise. No person has any authority to bind Medical Training Tools to any representation or warranty with respect to endoscopy accessories or investment goods.

Medical training tools isn't, under any circumstances, responsible for any damage created by and during use of the TD Box.

Use of the TD-box is at your own risk.